# ACE A

Atemalkoholtester

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Breathalyzer

Operating Manual – EN

Alcotest

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Alkohol Tester

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Test alcolico

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Alcoholímetro

Manual de instrucciones – ES

Andningsapparat

Bruksanvisning – SE

Alkomat

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# Atemalkoholtester ACE A

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## ACE A

### Important Notes – EN

Each breathalyzer which you buy from ACE Instruments can also be calibrated by ACE Instruments. Of course, we also calibrate devices which were bought from other contractors, as long as these devices are identical in construction to the devices sold by ACE Instruments. You can see the due date of the next calibration on the calibration sticker placed on your breathalyzer. If the sticker is missing, the calibration is to be done 6 months after the purchase date: e.g. purchase date: 13.05. = next calibration on the 13.11. and the next on the 13.05. of the following year.

For a calibration service you can send the device to the service address specified at [www.ace-technik.com](http://www.ace-technik.com).

#### **Why is a Calibration Necessary?**

The calibration is not mandatory, but the accuracy of the sensor installed in the breathalyzer will decrease steadily without a calibration and additionally guarantee expires, if you don't pay attention to the calibration ranges.

#### **Which Factors influence the Measurement Accuracy**

- The more often the device is used the more the zero point is influenced.
- The higher the alcohol concentration is, the more the zero point is relocated.
- Devices which are used rarely have a decreased sensibility, too.

ACE Instruments recommends using the breathalyzer at least once (1x) a week independently of having drunken alcohol or not. To maintain the measurement accuracy, the breathalyzer should also be calibrated periodically.

**Please also be aware of the Several Factors of Influence on the Measuring Result**

- Alcohol amount
- Period of the consumption of alcohol
- Gender
- Age
- Weight
- Height
- Drinking habits
- Health state
- Vitality of the liver
- Size of the lung
- Admission of food during the consumption of alcohol
- Influence of medicine

## **Important Notes on Test Execution and Self-assessment**

After the last sip of alcohol, you should wait at least 15 minutes, as there may still be residual alcohol in the oral cavity, which influences the measurement result. Also smoking should be avoided before a test, as cigarette smoke harms the sensor.

For each test you should use a new mouthpiece – not only for hygienic reasons, but also because an already used mouthpiece can influence the next measurement result. The mouthpiece could contain residual alcohol, which would affect the new test result when the same mouthpiece is used again (this for example could show an alcoholization of a person who hasn't drunken any alcohol).

If „FLOW“ or „FLo“ appears on the display, the given breath sample was insufficient or ended too soon. Blow into the mouthpiece for about 4 seconds constantly (without a break) – preferably as if you were whistling without a sound.

If the same person performs 2 tests consecutively, then wait at least 3 minutes between the two tests, as a too short waiting time may cause different measurement results. The air exhaled from the lungs is, after a test, no longer enriched with the same alcohol content as in the previous test. Therefore, an immediate second test will indicate a lower test result.

All measured values determined with this breathalyzer are always indicative and not legally binding.

### **Why the Estimated Alcoholization and the Measurement Result in most Cases cannot match**

An alcohol test is affected by a lot of different factors which also affect the test result. Beside the gender, age, body size and body weight, also the body fat percentage and the amount of body fluid have a big influence.

In addition, the general physical and mental state or the current daily constitution have to be taken into consideration. Furthermore, the meals eaten before a breath test (type of content and level of the stomach) and the intake of medication may influence the measurement.

Other influencing factors are, of course, the type of drink (percentage of alcohol content in the beverage and its carbon dioxide and sugar content), the drinking rate and the individual subject's duration of alcohol build-up or degradation in the body.

Some people quickly have a high alcohol content in the blood, with other people it takes longer until an alcoholization is detected. About 40-50% of the consumed alcohol is in the body within 15 minutes, after 30 minutes it is 65% and after 70 minutes the previously consumed alcohol is 100% absorbed in the body. However, it should also be noted that some people – partly due to genetic factors – quickly reduce the alcohol content in the blood, whereas others still have an increased alcohol content in the blood after a longer period of time.

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### Operating Manual – EN

#### **Author's Liability**

All information herein was created conscientiously and under the assumption of correctness. Hence, the author cannot be held liable for the use of any kind whatsoever and/or any ensuing consequences. Furthermore, the author is not liable for any costs or expenses for damage that result from any information in this document. Any information is subject to change without prior notice.

#### **Introduction**

Thank you for purchasing our ACE A breathalyzer! It measures the alcohol concentration in human breath using an electrochemical sensor and calculates the blood alcohol content.

The test results are very reliable and have a tolerance of  $\pm 0.075\text{‰}$  (per mill) with reference to  $1.00\text{‰}$  at  $25^{\circ}\text{C}$  ambient temperature. The measurement results are displayed as a 3-digit number with 2 decimals and are interpretable 1: 1 (for example the display shows the value 0.36 = 0.36 per mill).

With the measurement unit % BAC the test result is displayed with 4 digits and 3 decimals.

The fuel cell sensor inside the Breathalyzer can be calibrated. To maintain the measurement accuracy of this device and the sensor warranty, it should be calibrated regularly. For a calibration service, you can send the device to the service address specified at [www.ace-technik.com](http://www.ace-technik.com).

The device must only be used pursuant to the instructions mentioned in this manual. All measured values determined by this breathalyzer are only a guide and not legally binding. Breath tests should be carried out using clip-on mouthpieces (one new mouthpiece should be used per test for reasons of hygiene and to avoid falsified readings).

### **Note to country-specific Measurement Units**

Customers in the EU receive the breathalyzer with ‰ (per mill) as default setting, in the UK the default setting is mg/L, and in the USA, it is % BAC. In the chapter „Technical data,“ you will find information about the accuracy concerning the different measurement units. Please check the website of the Department of Transportation, the website of the Ministry of Transport, or automotive clubs to become informed about the current alcohol limits in your country.

### **Scope of Delivery**

- 1 Digital fuel cell, ACE A Breathalyzer
- 2 AAA batteries (1.5V, Alkaline)
- 3 Mouthpieces
- 1 Quick Start Guide (in German & English)
- 1 Manual (multilingual)

## Label of Individual Components

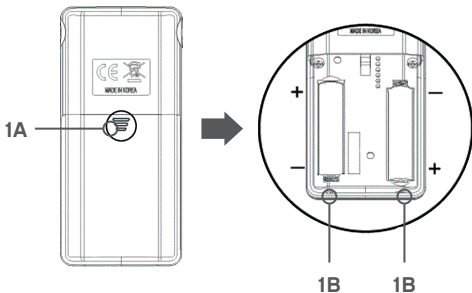
1. Mouthpiece and Mouthpiece slot
2. LCD Display  
(3 digits with unit ‰ and mg/L and 4 digits with unit % BAC)
3. Start button
4. Air outlet (always leave this part unblocked)
5. Battery compartment cover (at the back of the device)

### **Tips to Receive the Most Accurate Results**

- A)** Wait at least 15 minutes after your last sip of alcohol. Do not eat, drink, or smoke during that time.
- B)** Do not open the casing.
- C)** Do not keep the breathalyzer in cold, hot, moist, or badly ventilated places.
- D)** Keep the breathalyzer out of the reach of children.
- E)** Conduct three inlet tests after long storage before conducting the actual alcohol test.
- F)** Do not drop the breathalyzer.
- G)** Change the battery if it is depleted. Diminished battery capacity leads to incorrect test results.
- H)** Do not conduct tests in loud (sound waves disrupt the sensor), drafty, or poorly ventilated places.
- I)** After a test result of 1.00 per mill (per mill = EU setting) / 0.10% BAC (% BAC = US setting) / 0.50mg/L (mg/L = UK setting) or higher the sensor needs rest time. In this case wait at least three minutes before starting the next test.
- J)** Do not conduct tests in very cold, very hot, or an unventilated, smoky ambience.
- K)** Store the breathalyzer in a way to avoid penetration by dust or liquids.
- L)** Blowing too hard may cause damage to the sensor as saliva flows into it.

## Preparation for the Test

1. Push the battery cover (on the back of the device) downwards and open it (1A). Install the 2 AAA batteries (delivered together with the device) in the battery compartment while taking note of the battery polarity (1B). Close the battery cover again.
2. Insert a mouthpiece into the slot on the left front side of the breathalyzer. Ensure whether it is securely fitted. It is highly recommended to use a new mouthpiece for every test for sanitary reasons.

## Conducting the Measurement

1. Push the power button on the front of the device, then you hear one beep tone and shortly will see the number of tests performed with the device (Note: This number is only shown as long as you press the power button and if you press and hold down the power button for more than 5 seconds, the device will be turned off). If you don't press it anymore then you hear another beep tone and see "WAIT" on the display, additionally a countdown starts.
2. Wait until the end of the countdown you hear 2 beep tones and then you see "BLOW" blinking on the display. Additionally, you see 3 horizontal lines on the left side of the display.
3. As soon as you see "BLOW" on the display start blowing steadily into the mouthpiece for 4 seconds, the device will beep constantly during the test.

**Tip:** Blow like you would whistle without a tone (but without a break).

4. As soon as the beeping stops you hear a “clack” tone and a double beep tone (the test is finished). If the breath sample was not sufficient you will see „Flo“ on the display. Push the start button and start a new test.
5. After a successful test the device is analyzing the breath sample, you see several lines moving in clockwise circles around the display. After a few seconds the test result is displayed for 5 seconds (with 3 digits and 2 decimals) in ‰, then for 2.5 seconds in mg/L (= UK setting) and then for 2.5 seconds in % BAC (with 4 digits and 3 decimals). If the test result is higher than 0.50‰ or 0.25mg/L or 0.05% BAC, then you can hear a warn signal (beep sound) for about 10 seconds.

**Note:** The device will turn off automatically if it is not in use for about 3 minutes, additionally you hear a beep tone and on the display you see „Out“ and afterwards „OFF“. If you want to conduct a new test, please press the power button. You can turn off the device manually if you constantly press the power button for about 5 seconds, you shortly see „OFF“ on the display and the device turns off.

### **ACE Instruments recommends**

- Always use a new mouthpiece for each person and test that is conducted. ACE Instruments assumes no liability for any damages that may arise when several people use the same mouthpiece for multiple tests.
- Only use the provided individually packaged and hygienic mouthpieces. During each test the breathalyzer's internal and external parts come into contact with human secretion (saliva, blood, sweat, etc.)!
- Use safety gloves when pulling out a mouthpiece used by another person! You can buy appropriate gloves at ACE Technik ([www.ace-technik.de](http://www.ace-technik.de)).

## **IMPORTANT NOTICE**

WHEN YOU HAVE EATEN, DRUNK ALCOHOL OR, SMOKED A CIGARETTE, WAIT AT LEAST 15 MINUTES BEFORE STARTING THE MEASUREMENT. ANY TEST CONDUCTED PRIOR TO THE EXPIRATION OF THE RECOMMENDED WAITING PERIOD WILL RESULT IN INACCURATE RESULTS.

IF THE TEST RESULT IS BELOW 0.10‰ (PER MILL = g/L) OR 0.01% BAC (US setting) OR 0.05mg/L (UK setting) THEN THE DEVICE SHOWS 0.00 (‰ OR % BAC OR mg/L – depending on the setting) ON THE DISPLAY. THIS HAPPENS BECAUSE AT A LOW RANGE OF ALCOHOL CONCENTRATION IT IS POSSIBLE TO MEASURE OTHER MATERIALS IN YOUR MOUTH.

This breathalyzer is a high quality and sensitive measurement device. To maintain the accuracy of the sensor, it should regularly be used. If the device is not used for an extended period, the sensor can dry out. ACE Instruments recommends using the breathalyzer at least 2 times per month (even if you have not been drinking alcohol).

Due to changes in the product, the procedures stated can change. This can take place without prior notice or adaptation of the operating manual and is no cause for complaint, return or price reduction. The user is responsible for recognizing discrepancies between the operating manual and the actual handling of the breathalyzer and must react accordingly.

### **Only Use Original ACE Instruments Mouthpieces**

- Guarantees test result accuracy
- Guarantees hygiene of you and the device
- Guarantees full warranty of this device

### **ACE Instruments advises**

**even if you get a result meaning „safe range“, you should never drive after consuming alcohol or do any other activities that are severely impaired by alcohol consumption and might endanger your health and safety or the health and safety of others.**

## Display Indications

Display	Meaning	Action
	The battery is empty.	Replace with new batteries.
<i>BAt</i>	The testing cannot be performed because the battery is low.	Replace with new batteries.
<i>FLo</i>	The sampling volume or blow power is not enough to analyze the breath sample.	Press the start button, the countdown will start again, and "BLOW" will appear on the display. You can now perform a new test.
<i>Out</i>	If no breath sample was given within 3 minutes, the device turns off automatically.	Press power button to restart the countdown cycle and retest.
<i>Outc</i>	The testing cannot be performed because the device is turned on outside of the operating temperature.	Repeat the test after the unit has reached a temperature within the operation range (5°C to 40°C).
<i>Err2</i>	"Err2" states that fluid or saliva has entered the device and the sensor is unable to evaluate it.	Wait 20 minutes and then try again. If "Err2" is displayed for a long time, contact your seller for customer service.
<i>500</i>	The total number of breath tests performed has reached 500. The device should be calibrated promptly to maintain the warranty and accuracy.	Contact your seller for a calibration service. You can send the device for calibration to the service address specified at <a href="http://www.ace-technik.com">www.ace-technik.com</a> .

## **Calibration**

To maintain the accuracy of the breathalyzer, ACE Instruments recommends a calibration after 500 tests or approx. every 12 months (depending which incident happens first). For a calibration service you can send the device to ACE Instruments (see [www.ace-technik.com](http://www.ace-technik.com)). The calibration is not mandatory, but the accuracy of the breathalyzer will decrease steadily and additionally the warranty for the sensor expires.

**We recommend sending it as a parcel, so that in case of loss an inquiry or proof of delivery is possible.**

Due to changes in the product, the procedures stated can change. This can take place without prior notice or adaptation of the operating manual and is no cause for complaint, return or price reduction. The user is responsible for recognizing discrepancies between the operating manual and the actual handling of the ACE Instruments a Breathalyzer and must react accordingly.

## **Battery Notices**

The device associated with this operating manual contains 2 AAA Alkaline batteries (1.5 V). These batteries are non-rechargeable. The use of a rechargeable battery is not possible with this device and can damage it. If the included batteries are depleted, they must not be thrown away into household waste. Used batteries may contain harmful substances that can cause damage to the environment or your health. Please return all depleted batteries to electrical appliance stores, who collect them for free.

## **Battery exchange**

To replace the batteries, push the battery cover on the rear of the unit downwards and replace the discharged batteries with new ones. Be aware of the polarity when inserting the batteries (+/- pole). Close the battery compartment cover again. In the EU, the return of used batteries is mandatory by law. All depleted and rechargeable batteries will be recycled. This way valuable resources like iron, zinc and nickel can be retrieved and reused. Battery recycling is the easiest lesson in environmental protection – so please support it!

The shown garbage bins mean that batteries or rechargeable batteries must not be thrown into the household waste.

The signs below the garbage bins are acronyms for the chemical substances inside the battery:

Hg: Battery contains Mercury  
 Cd: Battery contains Cadmium  
 Pb: Battery contains Lead

#### Technical Data\*

Sensor	Fuel Cell Sensor
Accuracy	<b>+/- 0.075‰ (per mill) with reference to 1.00‰ at 25°C (per mill = default setting)</b> OR +/- 0.0075% BAC (Blood Alcohol Concentration) with reference to 0.100 % BAC at 25°C OR +/- 0.0375mg/l with reference to 0.50mg/l at 25°C
Detection Range	<b>0.00 up to 4.00‰ (‰ = default setting)</b> OR 0.000 up to 0.400% BAC (USA) OR 0.00 up to 2.00mg/L (UK)
Warm-up Time	Within 15 – 20 seconds at 0.5‰. This may vary depending on the value of per mill / BAC which was measured.
Dimension	109 x 51 x 18mm (L x W x H)
Weight	84g (battery included)
Power Supply	2 pieces of 1.5V AAA Alkaline batteries (= power for up to 1000 tests)
Working Temperature	5°C to 40°C
Storage Temperature	0°C to 40°C
Calibration	Every 12 months or after 500 tests. For a calibration service you can send the device to ACE Instruments (see <a href="http://www.ace-technik.com">www.ace-technik.com</a> ).

\*The technical data are subject to change without prior notice

for functional improvements.

### **Warranty and Liability**

ACE Handels- und Entwicklungs GmbH (ACE Instruments) provides a warranty of two years after purchase for material and processing errors of the product (see attached warranty and liability declaration) in all versions and designations; and a warranty of one year for the sensor. This warranty includes the electronics as well as the casing and sensor without sampling components. Components that inevitably degrade over time (such as fuses) are not part of this warranty. The warranty is valid from the date of delivery or from the date of first activation (always the shorter period), if not stated otherwise in the accompanying ACE Instruments product literature.

### **Limitation of Warranty and Liability**

#### **1. PRODUCT DESCRIPTION**

ACE Instruments rejects any further expressly stated or implied warranty, including the marketability and suitability for a determined purpose. This warranty especially excludes calibration. After changing the sensor, calibration is absolutely necessary. If an exchange of the sensor or a calibration is required for a product, the user may receive a cost estimate. Should the user reject the estimate, ACE Instruments will send the product back unrepaired/not calibrated and the product will be labelled accordingly. Should the product not fulfil the warranty, the claim of the buyer and the obligations of ACE Instruments are limited exclusively to the discretion of ACE Instruments for exchange, repair of the affected goods or reimbursement of the purchasing price for the faulty merchandise.

#### **2. APPLICATION RESTRICTION**

ACE Instruments is by no means liable for additional, incidental or consequential damage such as, among others, lost profit and downtime that result from selling, producing or using products that are part of this application restriction. This is independent of any claims emanating from, for instance, contracts, negligence or other culpable actions. The product, which is accompanied by this warranty and liability declaration (no matter the version or designation), in particular cannot be used in a legally binding manner to determine one's capability to operate vehicles or execute action that could lead to personal injury or injury to others under the influence of alcohol.

The thorough check for damage of all products upon receipt, calibration suitable for the given use by the buyer and the proper use, maintenance and service according to the instructions of the accompanying ACE Instruments product literature are mandatory prerequisites of this warranty. Maintenance and service work performed by non-qualified persons as well as the use of non-approved consumables or spare parts results in the voiding of this warranty. As for all technically challenging products, it is a fundamental prerequisite and mandatory for ACE Instruments warranty that all users of these product are familiar with the properties and restrictions of the devices as stated in the respective literature.

### **3. ADVICE RESTRICTION**

The buyer is solely responsible and confirms that they have checked the intended use and suitability of purchased goods for any purpose. Between the two parties it is expressly agreed that all technical or other advice given by ACE Instruments regarding the use of goods or services is free of charge and made under risk to the buyer exclusively. Therefore, no obligation or liability can result to ACE Instruments from any recommendation or their consequences.

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ACE Handels- & Entwicklungs GmbH