ACE

ACE XS

Breathalyzer

Operating instructions – EN

Atemalkoholtester

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Alcotest

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Alkohol Tester

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Test alcolico

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Alcoholímetro

Manual de instrucciones – ES

www.ace-technik.com

Breathalyzer ACE XS

Operating instructions – EN

Disclaimer

The information in these operating instructions has been compiled by ACE Handels- und Entwicklungs GmbH (ACE) with the utmost care. We assume that all information is correct. Nevertheless, we cannot accept any liability for any consequences arising from the use of this information. ACE is not liable for any costs or damages that may arise from the use of this manual. The contents of this manual are subject to change at any time without prior notice.

Parts and Accessories

Please check the entire contents for all parts and accessories. If anything is missing, please contact info@ace-technik.com, and we will offer you a replacement.

The scope of delivery includes

- ACE XS breathalyzer
- Key fob
- 5 mouthpieces
- Operating instructions

Please ensure that all the items listed are present before using the breathalyzer.

Introduction

Congratulations on your purchase of the ACE XS breathalyzer and thank you very much! You have made an excellent choice. The ACE XS is a high-precision, police-accurate breathalyzer in a practical key ring format. With its electrochemical sensor, it reliably measures the alcohol concentration in the breath and calculates an estimate of the blood alcohol concentration (BAC).

This device offers the best technology for compact breathalyzers and was specially developed for private users. Years of research and development have gone into the ACE XS to provide you with a reliable and user-friendly product. To ensure accurate and reliable results, please use the ACE XS only in accordance with the instructions in this manual.

The ACE XS provides test results with a measurement tolerance of ± 0.05 ‰ (per mille) at a reference value of 1.00 per mille and an ambient temperature of 25°C. The measuring unit ‰ (per mille) is the default setting in the EU.

The measurement results are displayed as a 3-digit numerical value with 2 decimal places and can be interpreted directly. For example, a displayed value of 0.36 means that the blood alcohol concentration is 0.36 per mille. Measured values in % BAC (Blood Alcohol Concentration) are displayed with 4 digits and 3 decimal places. The electrochemical sensor integrated in the breathalyzer guarantees precise and reliable measurements.

The electrochemical sensor integrated in the breathalyzer can be calibrated. To maintain the measuring accuracy of this device and the sensor warranty, we recommend regular calibration. Customers are welcome to send the device to the following address for calibration service:

The device may only be used in accordance with the instructions in the operating instructions. All measured values determined by this breathalyzer are for guidance only and are not legally binding. We recommend the use of clip-on mouthpieces for breath tests. To practice good hygiene and to avoid falsified readings, a new mouthpiece should be used for each test.

Important instructions for using the ACE XS breathalyzer

- A)** Wait at least 15 minutes after the last sip of alcohol until performing the alcohol test. Do not eat, drink or smoke during this time.
- B)** Do not open the external hardware.
- C)** Do not store the breathalyzer in a cold, hot, damp or poorly ventilated place.
- D)** Keep the breathalyzer out of the reach of children.
- E)** After a longer storage period, carry out three „run-in tests“ before carrying out the actual alcohol test.
- F)** Avoid dropping the breathalyzer.
- G)** Replace the battery if its capacity is low. A reduced battery capacity can lead to incorrect test results.
- H)** Do not carry out a test in noisy environments, as sound waves can interfere with the sensor. Also avoid draughty or poorly ventilated areas.
- I)** After a test result of 1.00 per mille or higher, the sensor requires a longer recovery time. In this case, wait at least three minutes until the next test.
- J)** Avoid testing in very cold, hot or smoky environments with poor ventilation.

- K) Store the breathalyzer in a place where it is protected from dust and liquids.
- L) Do not blow too hard into the device, as this can damage the sensor, especially if a lot of saliva gets into the device when blowing.
- M) Do not store the device near alcohol-based agents such as disinfectants or solvents and do not use them to clean the device.

Preparation for the test

1. Open the battery compartment cover on the top of the device using the notch. Insert the AAA battery (alkaline, 1.5 V), included with delivery, into the battery compartment. Make sure that the polarity (+/-) is correct when inserting the battery and when replacing it later.
2. Insert a mouthpiece into the opening on the right-hand side of the device. Make sure that the mouthpiece is firmly in place. Use a new mouthpiece for each test subject to ensure proper hygiene and maintain measurement accuracy, as a used mouthpiece may contain residual alcohol inside.

Carrying out the test

1. Press the power button for about 1 second until the device beeps and the ACE logo appears on the display. The device goes through a warm-up phase, which is then displayed. After the warm-up phase, the device beeps twice and „Blow“ appears on the display, indicating that the device is ready for the test.
2. Blow into the device through the mouthpiece. The device automatically detects the air flow and starts to beep continuously. Maintain the flow of breath for about 3-4 seconds. The breathalyzer ends the test with an audible and perceptible „click-clack“ sound. The result appears on the display.

Note

The device switches off automatically after about 10 seconds without performing any further tests. Before switching off, a beep sounds and „Power Off“ is shown on the display. Another beep is then emitted and the device switches off completely. You can also switch off the breathalyzer manually by pressing and holding the start button for 2-3 seconds. Shortly before switching off, you will hear a beep and „Power Off“ will appear on the display at the same time.

ACE recommends

- Always use a new mouthpiece for each person performing the test! ACE accepts no responsibility for possible damage caused by the use of one mouthpiece by several persons or for several tests.
- Only use hygienically individually packaged mouthpieces from ACE. The test device can become contaminated if shared by several people.
- Wear gloves when removing mouthpieces from the device after a test, especially if different people have carried out the test. ACE Technik (www.ace-technik.com) offers suitable gloves for this purpose.

Important notes

- If you have eaten, drunk alcohol, or smoked, wait at least 15 minutes before taking a measurement.
- This breathalyzer is a high-quality and sensitive measuring device. To maintain the measuring accuracy of the electrochemical sensor, the device should be used regularly. If the device is not used for a long period of time, the sensor may dry out. It is recommended to use the breathalyzer at least twice a month for alcohol testing, even if you have not consumed alcohol.
- For measured values below 0.07‰ (or 0.007% BAC or 0.035 mg/L - depending on the setting), the test result

is displayed as 0.00 (‰ or mg/L or % BAC - depending on the setting). This is because other substances may be present in the breath at low alcohol levels that could influence the result of the breath sample.

Only the use of original ACE mouthpieces to

- ensure the accuracy of the measurement results.
- guarantee zero cross contamination.
- secure the full device warranty.

Warning from ACE

Even if the device indicates an alcohol level in the „safe“ range, this does not mean that you can safely drive a vehicle or carry out other activities that could be dangerous under the influence of alcohol after consuming alcohol.

Switching the measuring units

After a test has been carried out, the breathalyzer displays the result in per mille as standard. However, by briefly pressing the power button, you can switch between different measurement units, including BAC (blood alcohol concentration in %) and mg/L (milligrams of alcohol per liter of breath). This allows you to display the measurement results in the unit that suits you best or that is common in your region.

Warnings on the display

- **Warm Up:** The appliance warms up. Please wait until this process is complete.
- **Blow:** The device is ready for use. Put on a mouthpiece and blow for 3-4 seconds.
- **Analyzing:** The test procedure is evaluated. The device analyzes the sample.
- **Blow Error:** The test was automatically interrupted. Please start a new test and blow continuously for 3-4 seconds until the test ends with an audible and perceptible click-clack sound.

- **Low Bat:** The battery is low. Please replace the battery.
- **CAL:** This message appears if a calibration is required. Please contact the ACE Service Center.
- **Err2:** This message indicates a sensor error. Please contact the ACE Service Center.
- **Err3:** This message appears if the operating temperature is too high. Move the device to a cooler place to restore normal operation.

Calibration

ACE recommends calibrating the integrated sensor approximately every 12 months or after 500 tests (whichever is sooner). Although calibration is optional, the accuracy of the sensor will decrease without regular calibration and the sensor warranty will be voided. You can send the device for calibration to the service address given at www.ace-technik.com.

We recommend sending the device as a parcel in order to provide proof of delivery or a follow-up request in the event of non-delivery. Please note that the procedures described may change due to product modifications without this being updated in the operating instructions. Such changes do not entitle the user to a complaint, return or price reduction. It is the user's responsibility to recognize deviations between the operating instructions and the actual operation of the breathalyzer and to act accordingly.

Notes on the batteries

The device referred to in these operating instructions is supplied with 1 AAA alkaline battery. The battery are not rechargeable. Do not use rechargeable batteries, as this may damage the device. Empty batteries must not be disposed of with household waste as they contain potentially harmful substances that can damage the environment and your health.

Please return the batteries to retailers or the recycling centers in your municipality. The return is free of charge and required by law. You should tape the terminals of lithium batteries before disposing of them.

How do you attach the key fob?

- Open the key ring using the snap mechanism.
- Remove the cover from the leather strap.
- Detach the leather strap from the metal piece.
- Thread the leather strap through the opening provided on the breathalyzer.
- Close the system by carrying out the steps in reverse order.

Battery change

To replace the battery, slide the battery compartment cover on the back of the device downwards and replace the discharged batteries with new ones. Make sure that the polarity is correct: the negative pole of the battery must be in contact with the springs in the battery compartment. Then replace the battery compartment cover on the back of the device and press it gently until it clicks into place.

Note on battery recycling

Recycling batteries is a simple measure to protect the environment and community health. Please do not dispose of old batteries and rechargeable batteries in household waste. They can be recycled, allowing valuable raw materials such as iron, zinc and nickel to be recovered.

Technical data

- Sensor type - Electrochemical sensor
- Battery - 1x alkaline AAA
- Battery life: approx. 100 tests per AAA battery
- Weight - 60g including battery
- Size - 61x51x44mm

- Measured value - 0.00 - 4.00‰, 0.00 - 2.00mg/L, 0,00 - 0,40%
- Possible deviation: 5%
- Operating temperature: 5°C - 40°C
- Storage temperature: 0°C - 60°C
- Mouthpieces: ACE article number 100086
- Key operation during the display of the measurement results: Short press (<0.5 seconds) --> Change the measuring unit (‰ --> mg/L --> %)
- Medium pressure (0.5 to 3 seconds) --> Trigger a new test
- Long press (3 seconds) --> device switches off.
- Recommended calibration: after approx. 1,000 tests

Warranty and liability

ACE Handels- und Entwicklungs GmbH (ACE) guarantees the product to which this warranty and liability declaration is attached - in all versions and designations - against defects in material and workmanship for a period of two years from the date of purchase and a one-year warranty on the sensor. This warranty covers the electronics and the external hardware but refers only to the sensor without sampling components. Parts that are inevitably destroyed during use (e.g. fuses) are excluded from the warranty. The warranty begins on the date of delivery or first use (whichever is shorter), unless otherwise specified in the literature accompanying the ACE product.

WARRANTY AND LIMITATION OF LIABILITY

Product restriction

ACE disclaims all other warranties, express or implied, including the implied warranties of merchantability and fitness for a particular purpose. In particular, the warranty does not apply to calibration. Calibration is mandatory after sensor replacement. If a sensor replacement and/or calibration is necessary, the user may receive a cost estimate. If the user rejects the cost estimate, ACE will return the product unrepaired/uncalibrated and mark it accordingly. If the product does not fulfill this warranty, the purchaser's claims and ACE's obligations shall be limited to the replacement or repair of the goods concerned or a refund of the purchase price for the goods complained about, at its own discretion.

Application restriction

In no event shall ACE be liable for any additional, incidental or consequential damages, including, but not limited to, lost profits and loss of use, arising out of the sale, manufacture or use of the products sold under this disclaimer, whether such claims arise in contract, negligence or other tortious action.

In particular, the product to which this warranty and liability statement is attached, regardless of the version and designation, cannot be used in a legally binding manner to determine the fitness of a person to drive a vehicle or to carry out an action which, under the influence of alcohol, results in danger to himself or others. Careful inspection of all products upon receipt for damage, calibration suitable for the purchaser's particular application and proper use, repair and maintenance in accordance with the

instructions in the product literature supplied by ACE are mandatory requirements for this warranty. Repair or maintenance by unqualified personnel will void the warranty, as will the use of non-approved consumables or replacement parts. As with all technically sophisticated products, it is a fundamental and mandatory requirement of ACE' warranty that all users of these products are familiar with the use, capabilities and limitations of the product as described in the applicable product literature.

Consultation restriction

The Buyer acknowledges that they have verified the intended use and suitability of the purchased goods for any purpose under its sole responsibility. It is expressly agreed between the parties that any technical or other advice given by ACE regarding the use of the goods or services is given free of charge and at the Buyer's sole risk. Therefore, ACE shall have no obligation or liability whatsoever arising from the recommendations or the results thereof.

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ACE Handels- & Entwicklungs GmbH